



Indications for use

- The OrthoCor Active Knee System is indicated for adjunctive use in the palliative treatment of post-operative pain and edema in superficial soft tissue.
- Temporarily relieves minor muscle and joint aches and pain associated with overexertion, strains, sprains, and arthritis.

Contraindications

- Do not use on patients with a cardiac pacemaker, cardioverter defibrillator, neurostimulator, infusion pump or any active medical implant.
- Do not use on patients who have ANY implanted metallic lead or any type of wire coil implant, or any implanted system that may contain a lead.
- Do not use on patients who are or may be pregnant.
- Do not use on patients with an open wound at the area of application.
- Do not use on patients not fully aware to the sensation of heat.
- Do not use on patients with poor circulation or heart disease.
- Do not use on patients with diabetes.
- Do not use on patients under the age of 18 or individuals with open bone growth plates.

Warnings

- WARNING: This device should be used under the continued supervision of a licensed health care practitioner.
- WARNING: THIS PRODUCT CAN CAUSE BURNS AND SKIN IRRITATION. Your risk of burning increases as you age. Check skin frequently during use for signs of skin irritation or burns.
- WARNING: Use carefully. Remove immediately if the knee becomes too hot or discomfort occurs.
- WARNING: Do not use with pain rubs, medicated lotions, creams or ointments.
- WARNING: Do not use with other forms of heat. This may cause burns.
- WARNING: Precaution should be taken prior to using this device on patients with neoprene allergies.
- WARNING: Precaution should be taken when using this device on the elderly or incapacitated persons. The unattended use of OrthoPods by these patients may be dangerous.
- WARNING: Precaution should be taken when using this device on patients with sensitive skin. If the patient experiences skin irritation, discontinue the use of the device and consult your licensed health care practitioner before using the OrthoCor Active Knee System again.
- WARNING: Equipment is not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide or oxygen enriched atmospheres.
- WARNING: Do not apply the OrthoPods directly to bare skin as it may cause burns.
- WARNING: Do not open or puncture the OrthoPods.
- WARNING: Do not let the contents of the OrthoPods make direct contact with your mouth or eyes. Immediately and thoroughly rinse with clean water, if contact with OrthoPods' contents occurs.
- WARNING: Do not ingest the contents of the OrthoPods. IMMEDIATELY CALL the nearest Poison Control Center if OrthoPods' contents are ingested.
- WARNING: Do not over-tighten the Knee OrthoCuff as this may restrict blood flow and result in injury.

Cautions

- CAUTION: Use this device only in the prescribed manner and for the prescribed diagnosis.
- CAUTION: KEEP AWAY FROM WATER. Use in dry environment.
- CAUTION: The device is not indicated for treatment of deep tissue such as internal organs.
- CAUTION: Do not use while the OrthoCor Active Knee System is charging.
- CAUTION: Do not machine wash the OrthoCor Active Knee System.
- CAUTION: Do not crush the OrthoSource or the Knee OrthoCuff as this may damage the electronics.
- CAUTION: Do not reuse the OrthoPods.
- CAUTION: Removing the OrthoPods will stop the treatment.
- CAUTION: The AC adapter must be disconnected from the OrthoSource for treatment to occur.
- CAUTION: Keep out of reach of children.
- CAUTION: For external use only.
- CAUTION: NEVER heat the OrthoCor Active Knee System or OrthoPods in the microwave as the system could catch on fire.